

Dear Doctor,  
Thank you for choosing a BabyNEXT™ test, please read the following instructions carefully to perform a proper DNA collection.

**ATTENTION!**

To properly perform the non-invasive DNA collection procedure by buccal swab, it is important to remind the patient of:

- Not to eat
- Not to drink
- Not to chew gums
- Not to brush his/her teeth

During the hour prior to the collection.

It may be helpful a short rinse with room temperature natural water before proceeding with the tampon rubbing.


NB: For infant patients, carry out the sampling at least 2 hours after feeding.

## COLLECTION PROCEDURE

1. Open the package.




2. Pull out the swab from the test tube.

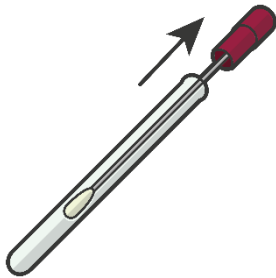

Attention! Do not touch the swab cotton part in order to avoid contamination!


**Eurofins Genoma Group S.r.l. a socio unico**

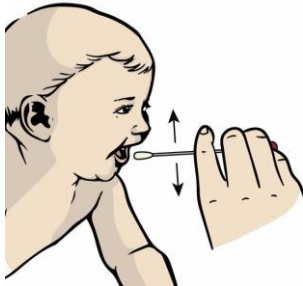

3. It is advisable to rub the swab on both cheeks, and the inner lining of the lips.

4. Repeat the action several times verifying that you have successfully completed the procedure.

4. For a standard collection it is necessary to rub for at least 60 seconds.

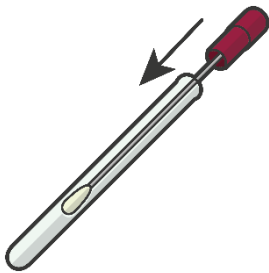

At the end of the collection, handle the swab carefully.

5. Reinsert the swab into the test tube

6. Close the swab carefully until you hear the final click that will preserve the sample during shipment

7. Apply the label with the patient's name on the test tube.

8. The collected DNA must be dispatched at room temperature in the parcel together with the authorization form.

9. Insert everything in the courier envelope and proceed to the shipment.